

BOWIE & DICK TEST PACK



Directions For Use

Bowie Dick Test Pack is used to determine steam penetration and leakage ability of the vacuum sterilizers operating at 134 °C – 3,5 /121 °C-15 minutes.

According to autoclave machine capacity test packs are produced as 200 sheet and 400 sheet.

The Bowie Dick Test should be carried out in autoclave every day.

The device is operated by adjusting Bowie Dick diagram in autoclave.

Color change

If colors of all lines change from blue to green the operation process is considered successfully.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each indicator and packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

BOWIE & DICK TEST PACK



Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Safety Precautions

Do not use damaged products.

Bowie & Dick Test Pack Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
Bowie & Dick Test Pack (400 sheets)	15.3	50	52	29	29
Bowie & Dick Test Pack (200 sheets)	15.3	100	52	29	29