

ETHYLENE OXIDE TAPE



Directions For Use

- 1-Cut tape according to your need
- 2-Wrape load..

Color change

After the sterilization process color changes from red to yellow.

Interpretation

Ethylene Oxide tape is a tape that is used for packaging the materials while placing them into the ethylene oxide sterilization.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each tape and packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

ETHYLENE OXIDE TAPE



Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

After wrapping process autoclave tape can be used to prevent crepe paper opening. Tape dimensions are 19 mm x 50 m and 25 mm x 50 m. The tape should be cut according to need. Do not use damaged products.

Ethylene Oxide Tape Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
Ethylene Oxide Tape (19x50)	6,5	30	28	28	30
Ethylene Oxide Tape (25x50)	8	30	28	28	30