

# PCD & HELIX TEST



## Directions For Use

- 1- Open the PCD/Helix Tube
- 2- Fold the indicator strips from the line
- 3- Place indicator inside the PCD/Helix Tube.

## Color change

After Steam sterilization process color changes light blue to Dark Green or tones.  
After EO sterilization process color changes red to yellow or tones.

## Interpretation

This test is used for Steam and EO sterilization process to challenge sterilization process. They have Helix/PCD, Type 6, Bowie & Dick and Ethylene Oxide indicator options. 1 box of Helix/PCD test contain 100 pcs or 400 pcs indicator according to demand.

## Storage

Keep products at room temperature 25°C

## Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

## Lot Number

Unique identification code, LOT is printed on each indicator and packaging labels.

## Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

# PCD & HELIX TEST



## Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

## Additional Information

Do not use damaged products.

### Helix/ PCD Test Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
Helix Test	3,25	50	52	29	29
PCD Test	3,25	50	52	29	29