


# PLASMA TAPE

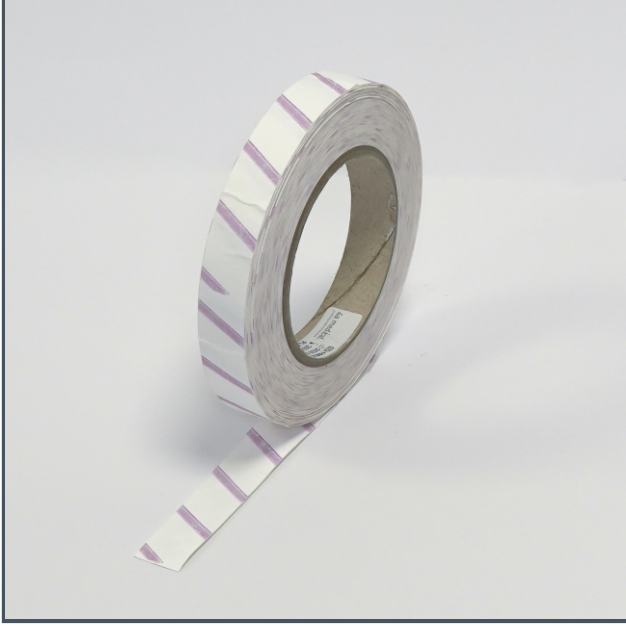

## Directions For Use

- 1-Cut tape according to your need
- 2-Wrape load..

## Color change

A color change is seen on the tape if the package is sterilized or not.

After sterilization process color changes from purple/red to blue.

## Interpretation

Plasma tape is a tape that is used for packaging the materials while placing them into the plasma sterilization.

## Storage

Keep products at room temperature 25°C

## Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.


## Lot Number

Unique identification code, LOT is printed on each tape and packaging labels.

## Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

# PLASMA TAPE

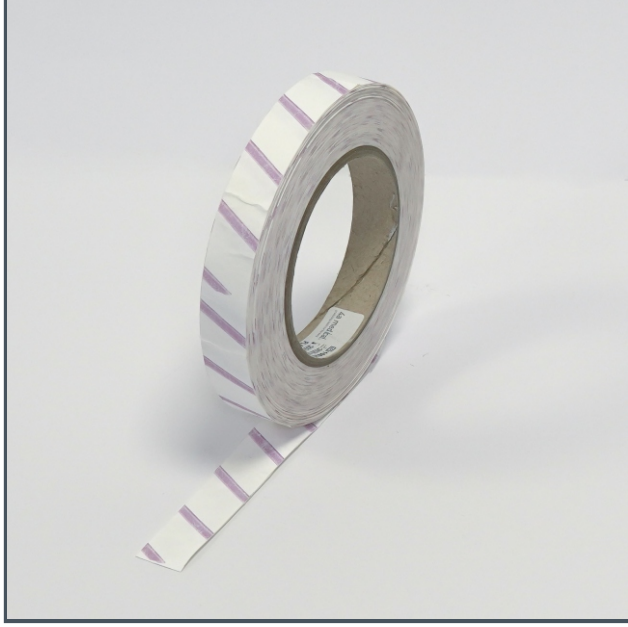

## Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

## Additional Information

After wrapping process autoclave tape can be used to prevent sms paper opening.

Tape dimension is 19 mm x 25 m.

The tape should be cut according to need.

Do not use damaged products.

## Plasma Tape Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
Plasma Tape (19x25)	8,25	30	28	28	30