

STEAM AUTOCLAVE TAPE



Directions For Use

- 1-Cut tape according to your need
- 2-Wrape load..

Color change

After sterilization process color changes from pink or red to brown.

Interpretation

The autoclave tape is a tape that is used for packaging the materials while placing them into autoclave. After wrapping process autoclave tape can be used to prevent crepe paper opening.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each tape and packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

STEAM AUTOCLAVE TAPE



Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Color change is seen on the strip if the package is sterilized or not.

Do not use damaged products

Tape dimensions are 19 mm x 50m, 25 mm x 50m.

The tape should be cut according to need.

Do not use damaged products.

Steam Autoclave Tape Packaging

| Product | Weight of Box (kg) | Pcs in a Box | Box Dimension(cm) | | |
|------------------------------|--------------------|--------------|-------------------|----|----|
| | | | | | |
| Steam Autoclave Tape (19x50) | 5 | 50 | 28 | 28 | 30 |
| Steam Autoclave Tape (25x50) | 5,5 | 40 | 28 | 28 | 30 |