

TYPE 5 INTEGRATOR



Directions For Use

- 1-Open pouch.
- 2-Pick an indicator strip.
- 3-Place strips in to pouch or load.

Indication for Use

Place unprocessed steam chemical integrator in each pack, pouch, container system or tray into the sterilizer. Process the load according to sterilizer manufacturer's instructions. After sterilization process check the integrator.

Interpretation

If the dark bar on the steam chemical integrator has entered the blue ACCEPT area, release the load. It means sterilization process is successful.

If the dark bar has not entered into the blue ACCEPT area, DO NOT release the load.

It means sterilization process is NOT successful and please check all conditions of machine and repeat the sterilization process.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each indicator and packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

TYPE 5 INTEGRATOR



Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Each chemical integrator is disposable. (single use only)

There are 100 integrators in type 5 integrator pack.

Do not use damaged products.

Shelf life is 4 years.

Type 5 Steam Chemical Integrator Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
Type 5 Steam Chemical Integrator	1.3	20	31	31	18