

TYPE 6 INDICATOR



Directions For Use

- 1-Open pouch.
- 2-Pick an indicator strip.
- 3-Place strips in to pouch or load.

Color change

Reference colors are mentioned on the strips for Type 6 Steam Chemical Indicator. If the indicator strip has not change to reference color, the goods should be returned to the sterilizer and subjected to a second treatment using a new indicator strip.

Interpretation

It has normal and self-adhesive options.

It is placed in the bundle, package and container and kept with patient records.

If the goods are wrapped with a pack, a control should be included in the center of the pack as well as on the out side

There are 250 indicators in each box. Each strip is single use only

There are two types as 134 degrees for 3,5min and 134 degrees for 7 min.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each indicator and packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

TYPE 6 INDICATOR



Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Be attentive to put chemical indicators in the midpoint of load. Inside the load, indicator part of chemical indicator strip should be looking upside.

If the goods enclosed in paper or plastic bags, put an indicator strip in each bag.

If the bags are opaque, one-two indicators should also be stuck out side selected bags inspected immediately after sterilization.

Do not use damaged products.

Type 6 Steam Chemical Indicator Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
Type 6 Steam Chemical Indicator 3,5 min.	11.5	100	30	35	27
Type 6 Steam Chemical Indicator 7 min.	12	100	30	35	27
Self Adhesive Type 6 Steam Chemical Indicator	7	40	31	31	18