

SMS PAPER



Directions For Use

The materials which is to be sterilized are placed into the wire basket.

If necessary, the indicator is placed into the wire basket.

Wire basket is wrapped by SMS paper.

After wrapping process plasma tape can be used to prevent SMS paper opening.

After sterilization process if any tear or damage on the product, please repeat process once more.

Interpretation

SMS paper is used for wrapping materials. It can be used for plasma sterilization process.

Storage

Keep products at room temperature 25°C

It can be transported on temperature between 5°C-55°C.

Shelf life of SMS paper is 5 years. The expiry date should be checked before use.

Expiry Date

The expiry date is printed on the packaging label.

Do not use after expiration date listed on the packs.


Lot Number

Unique identification code, LOT is printed on each packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the SMS paper or adversely affect the SMS paper performance.

SMS PAPER

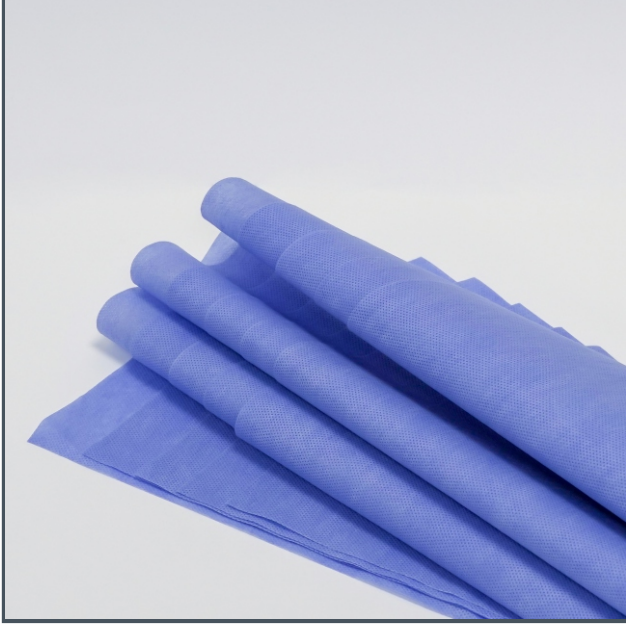

Release of Toxic Substances

The SMS paper releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

SMS paper is used for wrapping materials.
Do not use damaged products.

SMS Paper Packaging

Product	Weight of Box (kg)	Pcs in a Box	Box Dimension(cm)		
SMS Paper 40cm x 40cm	3	300	60	55	10
SMS Paper 50cm x 50cm	4.5	300	60	55	10
SMS Paper 60cm x 60cm	6	300	65	65	10
SMS Paper 75cm x 75cm	4	150	80	50	11
SMS Paper 100cm x 100cm	8.5	150	107	60	13
SMS Paper 120cm x 120cm	8	100	128	52	8