


FLAT REEL

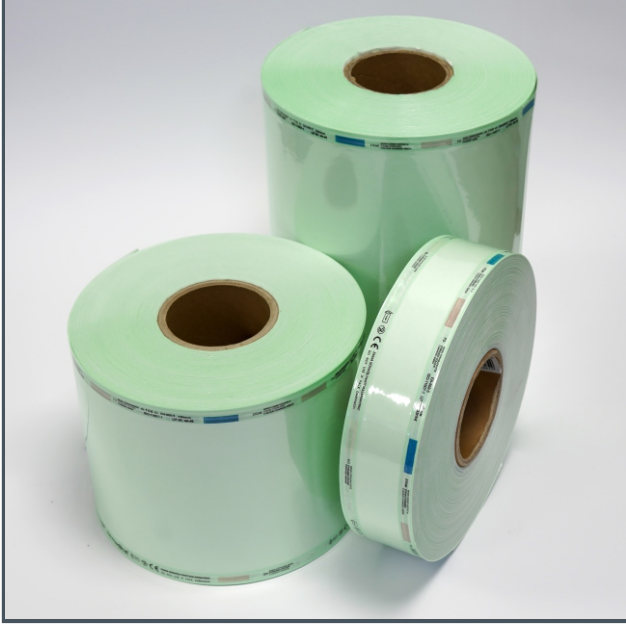

Indication for Use

The reels are cut with 5-7cm part from top and bottom. After cutting process, the product is placed into the reel. Top and bottom part will be opened. It is closed with suitable heat and sealing machine. If necessary, the indicator can be put into the reel. After that it will be ready for sterilization process. There are 2 or 3 indicators on the side of reels. Indicators are for steam, ethylene oxide and formaldehyde.

Color change

After sterilization color change as;
(Steam) Blue to Dark Green
(EO) Pink to Orange
(FO) Purple to Green

Interpretation

Reels are 5, 7.5, 10, 15, 20, 25, 30, 35 and 40 cm (+/- 1 cm) width and 200m. length (+/-1 m) . Gusset reels should be preferred for larger volumetric products. The sterilization process is made at 134°C for 3,5min or 7 min 121°C for 15min.

Storage

Keep products at room temperature 25°C

Expiry Date

Do not use after expiration date listed on the packs.


Lot Number

Unique identification code, LOT is printed on each reel.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the reel performance.

FLAT REEL

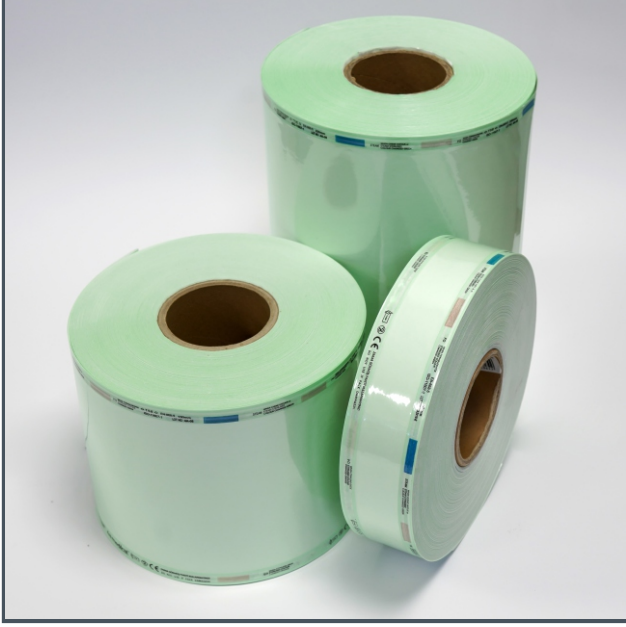

Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Please pay attention to the deformation and tear on the packs.

Do not overload the packs.

Please pay attention to the seal strength of both ends.

Do not use damaged products.

Flat Reel Packaging

| Dimension | Weight of Box (kg) | Pcs in a Box | Box Dimension(cm) | | |
|---------------|--------------------|--------------|-------------------|----|----|
| 5 cm x 200m | 12 | 10 | 42 | 23 | 27 |
| 7,5 cm x 200m | 14.5 | 8 | 42 | 23 | 32 |
| 10 cm x 200m | 19 | 8 | 42 | 23 | 42 |
| 12cm x 200m | 12.5 | 4 | 42 | 23 | 27 |
| 15 cm x 200m | 15 | 4 | 42 | 23 | 32 |
| 20 cm x 200m | 19.5 | 4 | 42 | 23 | 42 |
| 25 cm x 200m | 12 | 2 | 42 | 23 | 27 |
| 30 cm x 200m | 16 | 2 | 42 | 23 | 32 |
| 35 cm x 200m | 18 | 2 | 42 | 23 | 37 |
| 40 cm x 200m | 22 | 2 | 42 | 23 | 42 |