


# GUSSET POUCH

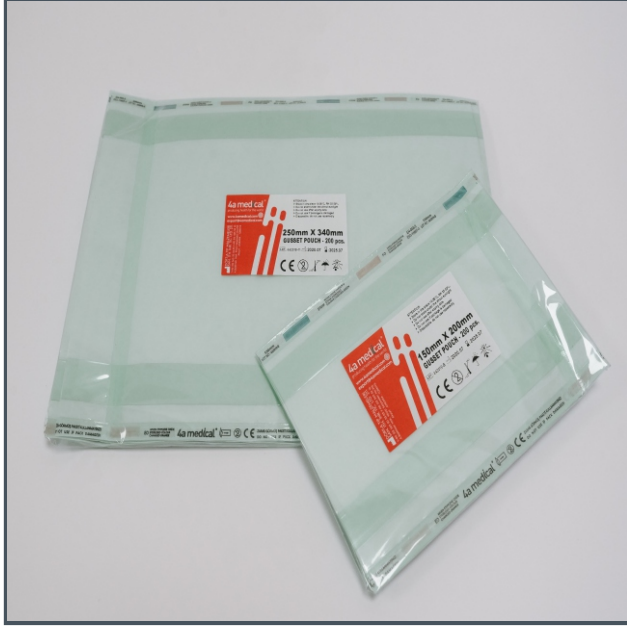

## Indication for Use

Pouches are cut and packed according to requested dimension. Medical instruments are placed into pouch. Opened part of pouch is closed with sealing machine. If necessary, the indicator can be put into the pouch. After that it will be ready for sterilization process. There are 2 or 3 indicators on the side of pouches. Indicators are for steam, ethylene oxide and formaldehyde.

## Color change

After sterilization color change as;  
(Steam) Blue to Dark Green  
(EO) Pink to Orange  
(FO) Purple to Green

## Interpretation

The sterilization process is made at 134°C for 3,5min or 7 min 121°C for 15min.

## Storage

Keep products at room temperature 25°C

## Expiry Date

Do not use after expiration date listed on the packs.


## Lot Number

Unique identification code, LOT is printed on each pouch and packaging labels..

## Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the reel performance.

# GUSSET POUCH

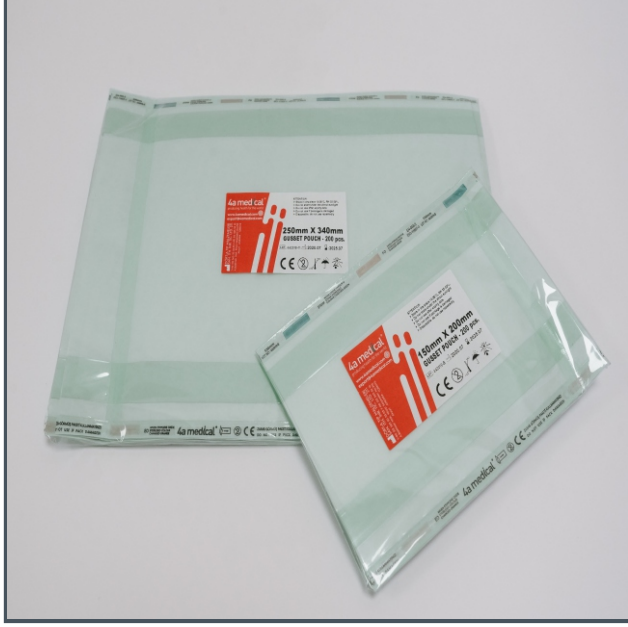

## Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

## Additional Information

After sterilization process if pouch damaged, please repeat the sterilization process.

Please pay attention to the deformation and tear on the packs.

Do not overload the packs.

Please pay attention to the seal strength of both ends.

Do not use damaged product