


SELF SEALING POUCH

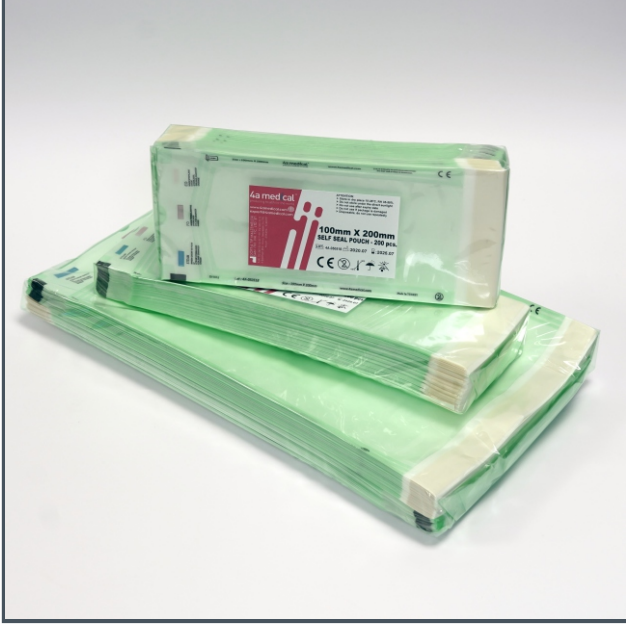

Indication for Use

Pouches are cut and packed according to requested dimension. If necessary, the indicator can be put into the pouch. Open part has self-sealing tape so you do not need to seal the pouch with sealing machine. Before peel to pouch check indicators on it and peel to pouch according to peel direction. After placing the medical instruments, you can close the pouch. After that it will be ready for sterilization process. This process is provided convenience and gain time. There are 2 or 3 indicators on the side of pouches. Indicators are for steam, ethylene oxide and formaldehyde.

Color change

After sterilization color change as;
(Steam) Blue to Dark Green
(EO) Pink to Orange
(FO) Purple to Green

Interpretation

The sterilization process is made at 134°C for 3,5min or 7 min 121°C for 15min.

Storage

Keep products at room temperature 25°C

Expiry Date

Do not use after expiration date listed on the packs.


Lot Number

Unique identification code, LOT is printed on each pouch.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the reel performance.

SELF SEALING POUCH

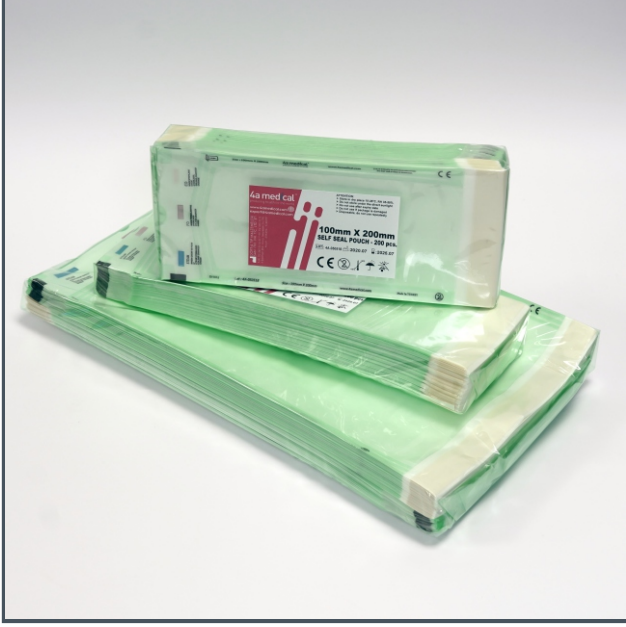

Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Please pay attention to the deformation and tear on the packs.

Do not overload the packs.

Do not use damaged product

Please make sure that self-sealing tape is sealed properly.