

PLS WASHER TEST



Directions For Use

PLS Washer test is used to control whether cleaning process is successful or not.

Surgical devices are placed into the wire basket before washing process.

- 1- Place PLS Washer test indicator in holder. Make sure the holder is clean and dry.
- 2- Holder should be placed vertically
- 3- Place empty basket with holder into a rack with load for processing.
- 4- After washing process Carefully remove the All Clean Test Indicator Strip from the holder. Inspect by placing the plastic film against a white background. Compare the results against the samples below to determine a pass or fail. Strip remains visible to the naked eye - the results is a FAIL.

Impingement Related Failure

- 1) Incorrect positioning of indicator
- 2) Blocked spray arms
- 3) Lack of water pressure
- 4) Over loading of rack

Chemistry Related Failure

- 1) Incorrect positioning of indicator
- 2) Enzyme detergent cycle is too short
- 3) Temperature parameters are not accurate
- 4) Chemistry injection rates are incorrect

Each chemical washer test is disposable. (single use only)

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the label. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each packaging labels.