

# PROTEST



## Directions For Use

Cotton swab is removed from the pack. Cotton swab is got wet by water vials which comes with box as 4 pcs. Protein residue is received from surgical devices surface. Protest is removed from the box and cotton swap is placed into the tube. Tube is shaken ten second to get result

## Color change

If chemical color turned purple, it means washing process is not successful. That means residual contain protein.

## Storage

Keep products at room temperature 25°C

## Expiry Date

The expiry date is printed on the product. Do not use after expiration date listed on the packs.

## Lot Number

Unique identification code, LOT is printed on each packaging labels.

## Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

## Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

## Additional Information

Protest is used to control whether cleaning process is successful or not.

After washing process, there can be some residuals on the surgical devices.

It can be rust, water drops, blood residual.

Each chemical protest is disposable. (single use only)

Do not use damaged products.

**4a medical**<sup>®</sup>  
producing health for the world

• Balıkhisar Mahallesi  
Köyüçi Serpmeleri No: 795  
Akyurt 06750 Ankara Türkiye

• export@4amedical.com  
sales@4amedical.com  
logistic@4amedical.com

• +90 312 363 50 52  
+90 312 363 50 53  
www.4amedical.com