


SONICONTROL

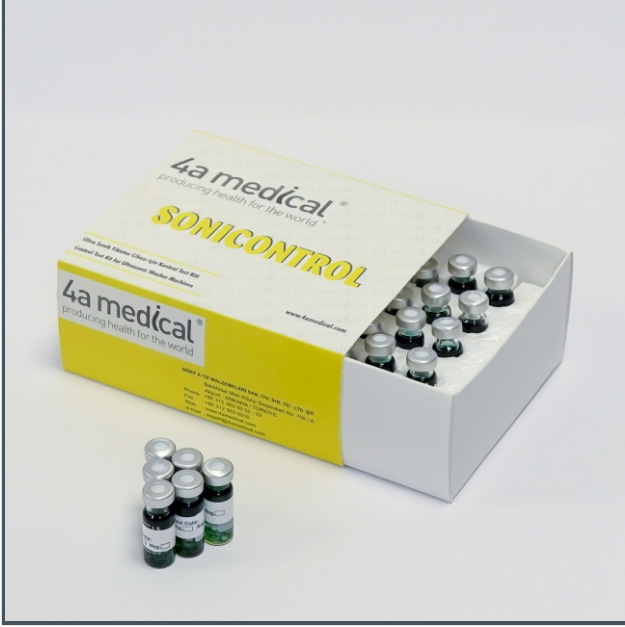

Directions For Use

Sonicontrol is a product which evaluates the performance of Ultrasonic machines.

Surgical devices must be washed after using, then you can put them into the ultrasonic machine.

Color change

Before the process, the colors can be green, light green or blue. They are different color because of the Ph. difference. After the cleaning process, the color must be yellow.

If there is a problem in the machine, Sonicontrol does not change the color.

The color changes from green to yellow.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product label. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Each sonicontrol is disposable. (single use only)

Do not use damaged products.