

WASHER TEST



Directions For Use

Washer test is used to control whether cleaning process is successful or not.

Surgical devices are placed into the wire basket before washing process.

Washer test has own plastic frame, so no need to use any other frame or boxes.

Washer test is removed from the aluminium foil.

It is placed into the wire basket. It is sent to the washing process.

If there is any residue on the washer test after washing process, it means that washing process is not successful and it should be repeated. If there is no residue it means washing process is successful.

Storage

Keep products at room temperature 25°C

Expiry Date

The expiry date is printed on the product label. Do not use after expiration date listed on the packs.

Lot Number

Unique identification code, LOT is printed on each packaging labels.

Interfering Substances or Conditions

There are no interfering substances or conditions that could affect the intended use of the indicator or adversely affect the indicator performance.

Release of Toxic Substances

The indicator releases no toxic substances in sufficient quantities to cause either a health hazard or a hazard to the intended properties of the product being sterilized during or after the sterilization process.

Additional Information

Each chemical washer test is disposable. (single use only)

Do not use damaged products.